

## DIRECTIONS FOR USE

Masket Design, LLC

**Description:** The MASK-it Eye Patch is designed as a translucent, single-use, adjustable, hygienic and adhesive paper patch.

**Indications for Use:** The paper patch is applied to, and worn by an individual for the execution of a variety of specific activities related to the need for temporary monocular occlusion.

**How supplied:** The patches are removed individually from a box with a roll of either 250 or 500 patches per box.

### How to Use (see reverse side):

- Gently peel the patch from the backing, starting from the smaller end of the patch – notice where the adhesive is located
- Flip the patch so the adhesive is face down – try to not touch the adhesive much
- Separate the paper at the partial pre-cut slit section of each patch
- Create a slight conical shape of varying depth, as needed. *Notice that the narrower the face, or the longer the lashes, the deeper the cone, and the flatter the face, the shallower the cone.* A shallower cone insures the best adhesion – there's no need to make the cone very deep
- Apply to the individual face by tapping on the glued smaller section between the eyebrows and the glued larger area near the cheekbone, below the eye – it is applied on an angle
- For the second eye, simply remove the patch and repeat the same procedure on the fellow eye, using the same patch

## DIRECTIONS FOR USE

Masket Design, LLC

